

FLUOROSIS

DIAGNOSIS & PRE-TREATMENT

Diagnose depth of lesion

Bleaching/Whitening if recommended by doctor (must wait 14 days)



Apply rubber dam/ liquid dam (for all lesion types)

Microabrasion, using a diamond bur, or sandblasting to target the specific area where the lesion exists without compromising the whole tooth structure

Now we can begin the Etching & Resin Infiltration...

ACCESSING THE LESION & INFILTRATION

Apply Icon-Etch

2 min

Apply Icon-Dry

30 s

Spots have diminished

Spots are still visible

Up to 2 more times

Repeat

Apply Icon-Infiltrant

at least 3 min

Light-cure all sides

40 s

Light-cure all sides

40 s

Reapply Icon-Infiltrant

1 min

Rinse thoroughly

Polish